



Product Overview Modern Wound Care

www.modernwoundcare.com

key

wound



superficial wounds
deep wounds

wound phase



necrotic wound
fibrinous coated wound
granulating wound
epithelising wound

exudation



low exudation
moderate exudation
heavy exudation

used for



primary dressing
secondary dressing
wound filler

Manufacturer:
TRIGOCare International GmbH
51674 Wiehl / Germany
tel: +49-(0)2262-72701-0
info@trigocare.com
www.trigocare.com

All rights reserved. No reproduction or use of text or pictures without the permission of TRIGOCare International GmbH. TRIGOCare International GmbH gives no warranty whatsoever for the up-to-dateness, accuracy, completeness or quality of the information provided. TRIGOCare International GmbH shall not be held liable for any loss or damage of a material or non-material nature caused by the use or non-use of the information presented or by the use of inaccurate or incomplete information.

Issue: 09/2023

©TRIGOCare International GmbH 2011-2017

Content:

About us	3
TRIGO PHMB	4
TRIGOtüll komb / TRIGOtüll komb adhesive	5
TRIGOpad aqua / TRIGOpad aqua comfort	6
TRIGO Superabsorber	7
TRIGOcolloid foam	8
TRIGOcolloid thin / TRIGOcolloid comfort	9
TRIGOfilm comfort	10
Become active	11
At a glance	12

Putting people first For us this means providing customised diagnostic and wound treatment with a carefully designed product range. We take into account patients' needs and all accompanying factors, with the aim of ensuring efficient and successful diagnostic & therapy. The use of effective products to promote an ideal moist wound environment is crucial in achieving this objective.

This multidisciplinary task entails not only medical and nursing challenges, but also the requirement to achieve cost efficiency. Selecting effective products helps to reduce treatment times and make cost savings, ultimately contributing to an improved quality of life for patients and their families.

You can have complete confidence in our high quality standards when making your choice of products. TRIGOCare International GmbH offers an advanced, phase-oriented range to provide you with reliable support at every stage of wound treatment. Our products are combinable and cover a wide range of applications.

Are you interested in TRIGOCare products?

Do not hesitate to contact us in case of any doubts. We will return to you promptly.

Phone: +49 (0)2262-72701-0


Email: info@trigocare.com

www.trigocare.com



TRIGO PHMB

sterile, moist wound dressing with polyhexanide (PHMB)

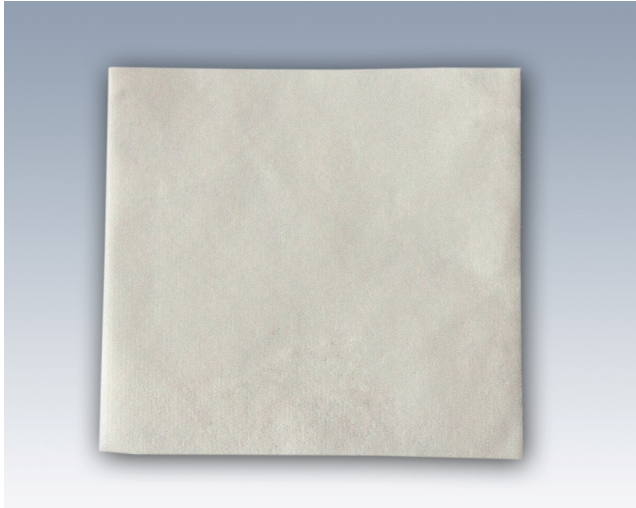

TRIGO PHMB is a sterile, moist non-woven wound dressing with polyhexanide (PHMB). The solution is used for decontamination and infection prophylaxis against microorganisms and provides preventive protection against bacterial and fungicidal pathogens in acute and chronic wounds. The particularly effective combination of ingredients of TRIGO PHMB dissolves the biofilm, cleanses the wound and creates an optimal wound environment for the formation of new granulation tissue. Must be fixed with a secondary dressing.


Advantages of TRIGO PHMB:

- Decontamination and infection prophylaxis against microorganisms
- preventive against bacterial and fungicidal pathogens in acute and chronic wounds
- Dissolves the biofilm
- Prevents the multiplication of germs and bacteria

Indications:

acute and chronic wounds at risk of infection:

- Postoperative wounds
- Ulcus cruris (lower leg ulcer)
- Superficial and deep wounds
- Diabetic foot syndrome
- Burns up to grade 2
- Pressure ulcer up to grade 3

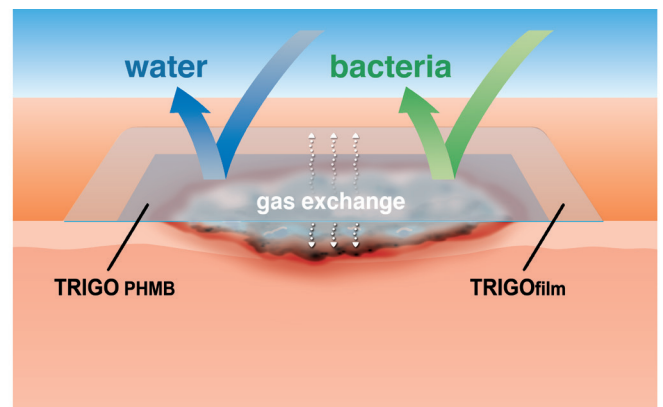

Order information



item-N°	size	DU
19010	10 x 10 cm	10 pcs.
19050	10 x 10 cm	5 x 10 pcs.

Advice for users

wound	wound phase	exudation	product type
			PD

TRIGOtüll komb / TRIGOtüll komb adhesive

Lipocolloid absorbent dressing

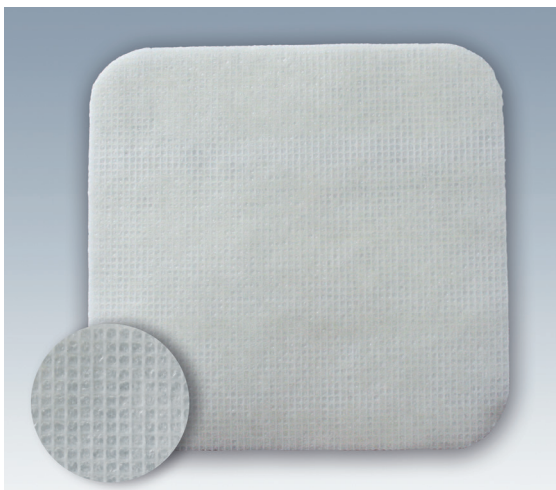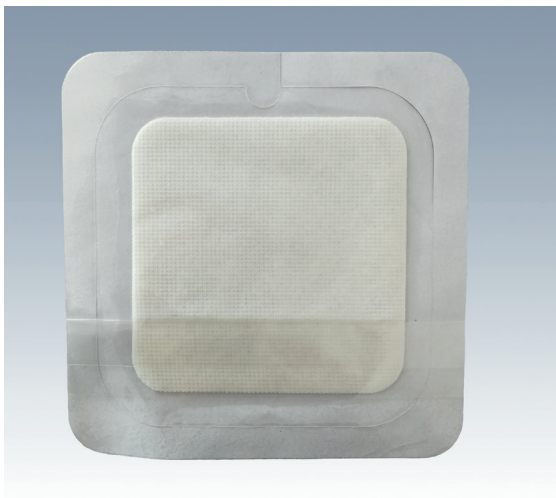

TRIGOtüll komb is a non-adherent, breathable wound dressing made of absorbent polyester fabric impregnated with a lipocolloid layer. The lipocolloid ensures that a soft gel is formed on contact with moisture, which keeps the wound environment moist, promotes wound healing and prevents adhesion to the wound dressing. The polyester fabric with super-absorbent powder absorbs the exudate quickly and reliably. Exudate is thus specifically drained away from the wound without drying out the wound, thus the risk of maceration can be significantly minimised. TRIGOtüll komb makes exudate management simple and user-friendly while providing a high level of patient comfort. This wound dressing must be fixed with an adequate secondary dressing.

TRIGOtüll komb adhesive has a self-adhesive, transparent border of 2.5cm.

Advantages of TRIGOcolloid foam:

- combines the positive characteristics of a hydrocolloid with the advantages of an absorbent wound dressing
- creates ideal moist wound conditions
- does not adhere to the wound
- atraumatic dressing change

Indications:

- low to moderate exuding wounds
- Burns
- Traumatic wounds
- Abrasions
- Post-operative wounds
- Leg ulcers
- Pressure ulcers
- Diabetic foot ulcers
- Epidermolysis bullosa

Order information TRIGOtüll komb

item-N°	size	DU
13010	10 x 10cm	10 pcs.
13050	10 x 10cm	50 pcs.

Order information TRIGOtüll komb adhesive



item-N°	size	compress	DU
13109	10 x 10cm	7 x 7cm	50 pcs
13110	15 x 15cm	10 x 10cm	10 pcs
13150	15 x 15cm	10 x 10cm	50 pcs

Advice for users

wound	wound phase	exudation	product type
			PD

TRIGOpad aqua / TRIGOpad aqua comfort

transparent hydrogel dressing

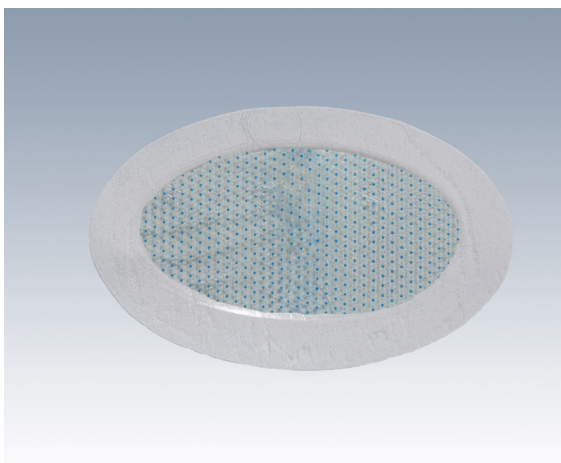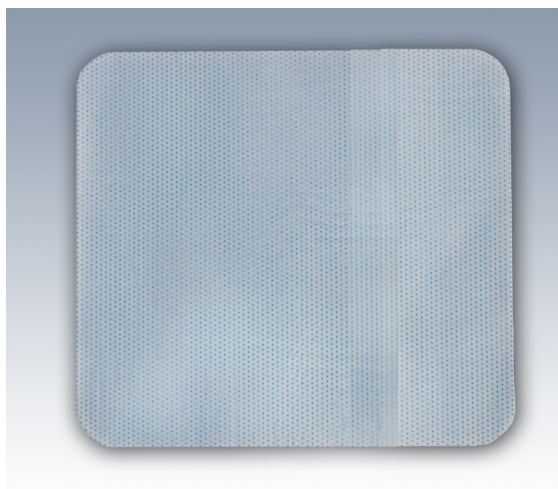

TRIGOpad aqua is a slightly cooling, transparent hydrogel dressing made of a PU gel (60% water content) with a gas-permeable but liquid- and germ-proof top layer made of polyurethane. The gel can both release moisture and absorb secretions. This creates a moist wound environment that promotes the natural wound healing process.

TRIGOpad aqua does not stick to the wound and can be removed without leaving any residue. The wound dressing is transparent and thus allows the wound to be observed without changing the dressing.

TRIGOpad aqua must be fixed with a secondary dressing, while TRIGOpad aqua comfort is self-adhesive.

Advantages of TRIGOpad aqua:

- releases moisture to the wound with simultaneous absorption of exudate
- Suitable for wound debridement
- does not stick to the wound
- ideal moist wound environment
- repellent to bacteria and moisture
- permeable to gases

Order information TRIGOpad aqua

item-N°	size	DU
11010	10 x 10cm	10 pcs.
11020	10 x 10cm	20 pcs.
11050	10 x 10cm	50 pcs.



Order information TRIGOpad aqua comfort

item-N°	size	DU
11030	68 x 43mm	50 pcs.

Indications:


- Wounds in the granulation and epithelialisation phase with little or no exudation
- Suitable for use in debridement

Advice for users

wound	wound phase	exudation	product type
			PD

TRIGO Superabsorber

super absorbent wound dressing

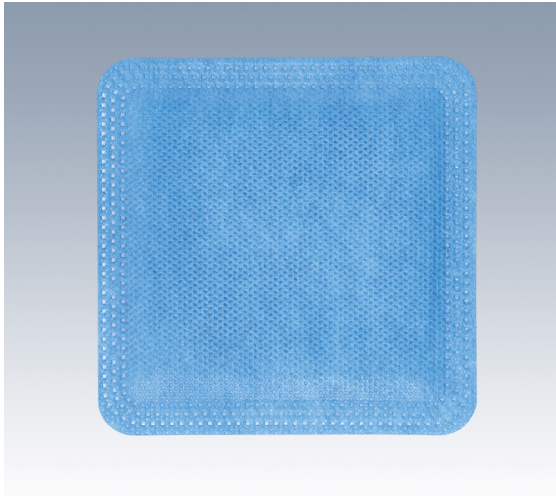

TRIGOfoam adhesive is permeable to gas but not to liquids and bacteria. The absorbent layer transfers exudates away from the wound and captures it without leaks or meltdown. The dressing does not adhere to the wound thereby reducing any pain associated with dressing removal preventing trauma to newly formed tissue.

TRIGOfoam adhesive is provided with an surrounding adhesive boarder so no secondary dressing is needed.

Advantages TRIGO Superabsorber:

- high absorption capacity and safe containment of the wound exudate
- minimizes the risk of maceration
- outer layer is impermeable for bacteria and liquids
- creates an ideal moist wound environment
- can also be used as part of compression therapy




Order information TRIGO Superabsorber

item-N°	size	DU
12050	10 x 10cm	25 pcs.
12051	20 x 10cm	25 pcs.
12052	30 x 20cm	25 pcs.

Indications:


- moderate to heavily exuding wounds
- leg ulcers
- pressure ulcers
- diabetic foot ulcers
- dehisced surgical wounds

Advice for users

wound	wound phase	exudation	product type
			PD, SD

TRIGOcolloid foam

sterile, self-adhesive hydrocolloid-foam-dressing

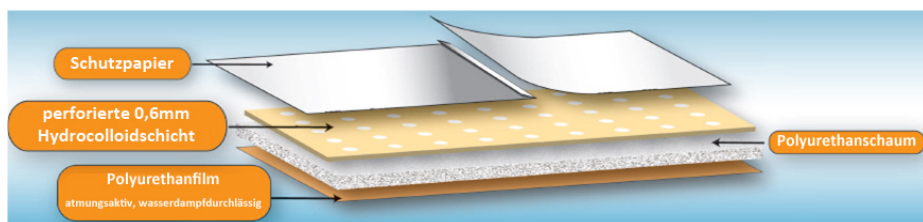

Advantages of TRIGOcolloid foam:

- combines the positive characteristics of a hydrocolloid with the advantages of a polyurethane foam
- good absorption capacity
- atraumatic dressing change
- self-adhesive but does not adhere to wound
- ideal moist wound environment
- bacteria- and moisture-resistant, but gas-permeable
- low macerations risk due to an excellent exudate management

Indications:

- low to moderate exsuding wounds
- Decubitus
- Ulcus Cruris of different origin
- Diabetic foot ulcers
- up to 2nd degree burns
- post-operative wounds
- split-skin grafts
- abrasions

TRIGOcolloid foam is a sterile, self-adhesive hydrocolloid dressing with a polyurethane foam layer. The hydrocolloid layer is perforated and enables the overlaying polyurethane foam a fast absorption of the wound exudate. So the exudate is piped away from the wound without itself running dry. Thereby the risk of maceration is minimised.

The hydrocolloid matrix transforms to a soft gel by contact with wound exudate which maintains a moist wound environment especially for low exuding wounds and thereby supports a faster healing. TRIGOcolloid foam is user friendly, facilitates the exudate management and increases the patient comfort at the same time.

Order information



item-N°	size	DU
14102	10 x 10cm	10 pcs.

Advice for users

wound	wound phase	exudation	product type
			PD, SD

TRIGOcolloid thin / TRIGOcolloid comfort

hydrocolloid wound dressing

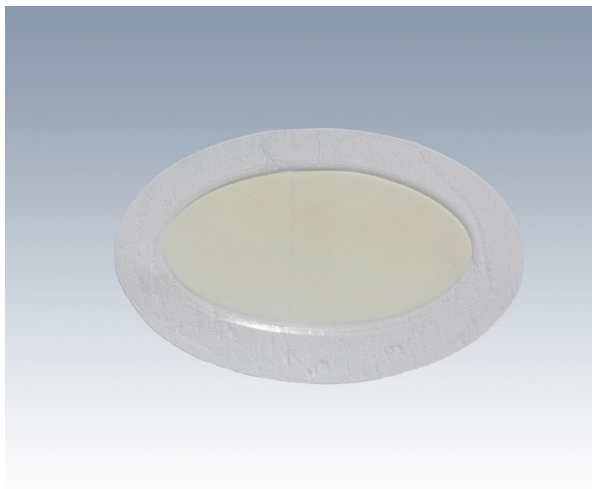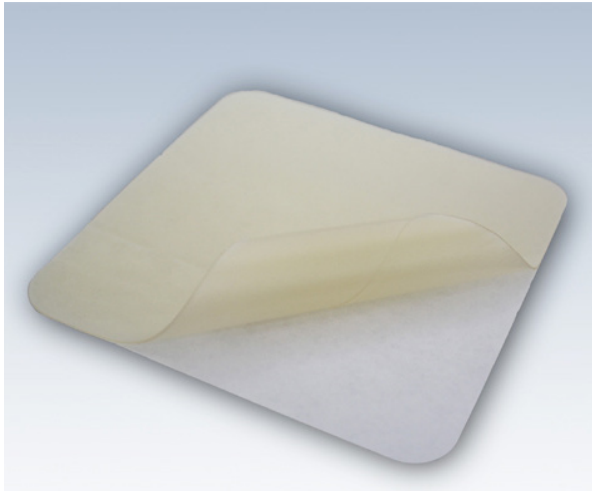

Advantages of TRIGOcolloid:

- does not adhere to wound
- ideal moist wound environment
- bacteria- and moisture-resistant, but gas permeable
- self-adhesive
- slightly transparent (easy wound surveillance)

TRIGOcolloid products are hydroactive, sterile, self-adhesive wound dressings for moist wound treatment.

The wound dressings maintain an ideal moist wound environment that promotes physiological healing processes. They are permeable to gases but not to liquids and bacteria. Due to the gel formation on contact with wound exudate, both wound dressings enable atraumatic dressing changes.

TRIGOcolloid thin is very easy to shape and, due to its high flexibility, is particularly suitable for use in the joint area and in exposed areas. The low thickness (0.6 mm) also allows good observation of the wound.

Order information TRIGOcolloid thin

item-N°	size	DU
15002	10 x 10cm	10 pcs.
15052	10 x 10cm	50 pcs.

Order information TRIGOcolloid comfort




item-N°	size	DU
15005	68 x 43mm	50 pcs.

Indications:

(depending on size of the wound, see instructions for use for details)

- Partial and full thickness dermal ulcers
- Pressure, leg and diabetic ulcers
- Burns (first and second degrees)
- Skin donor sites and skin grafts
- Superficial wounds/minor abrasions
- Protection of fragile skin
- Ostomy wounds to protect the skin from stomal effluent
- Post-operative dressings for suture line protection
- Infants diaper rash

Advice for users

wound	wound phase	exudation	product type
			PD

TRIGOfilm

transparent wound dressing on a roll



TRIGOfilm is a waterproof wound dressing coated with an acrylate adhesive. The wound dressing is transparent and thus enables the observation of the wound without dressing change. TRIGOfilm is permeable to gas exchange but impermeable to water and bacteria.

A grid division enables individual cutting.

Advantages of TRIGOfilm:

- very flexible
- waterproof, can be worn in the shower
- atraumatic dressing change
- allows easy observation of the wound
- ideal moist wound environment
- can be used to protect the wound edges
- bacteria- and moisture-repellent, but gas permeable




Indications:

- fixation of catheters and cannulas
- fixation of primary wound dressings (e.g. TRIGOpad aqua, TRIGO PHMB)

Order information TRIGOfilm





















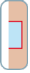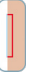






















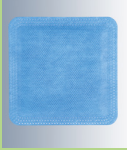



item-N°	size	DU
16020	10m x 5cm	1 roll
16021	10m x 10cm	1 roll
16022	10m x 15cm	1 roll

Advice for users

wound	wound phase	exudation	product type
			PD, SD

Become active



Products	   	wound phase    	wound  = deep  = superficial	exudation  low  moderate  heavy	secondary dressings needed 	combination possibilities 	kind of dressing
TRIGOfilm							TRIGOpad aqua TRIGO PHMB TRIGOtüll komb TRIGOSuperabsorber PD, SD
TRIGOtüll komb / tüll komb adhesive						TRIGOfilm	PD
TRIGOcolloid thin							PD
TRIGO colloid foam						TRIGO PHMB	PD, SD
TRIGO Superabsorber						TRIGOfilm	PD, SD
TRIGOpad aqua						TRIGOfilm	PD
TRIGO PHMB						TRIGOfilm	PD

At one glance

Item-N°	Description	DU
Sterile moist wound dressing with polyhexanide (PHMB)		
19010	TRIGO PHMB, 10cm x 10cm	10 pcs.
19050	TRIGO PHMB, 10cm x 10cm	50 pcs.
Lipocolloid dressing		
13010	TRIGOtüll komb, 10cm x 10cm	10 pcs.
13050	TRIGOtüll komb, 10cm x 10cm	50 pcs.
13109	TRIGOtüll komb adhesive, 10cm x 10cm	50 pcs.
13110	TRIGOtüll komb adhesive, 15cm x 15cm	10 pcs.
13150	TRIGOtüll komb adhesive, 15cm x 15cm	50 pcs.
Transparent hydrogel dressing		
11010	TRIGOpad aqua, 10cm x 10cm	10 pcs.
11020	TRIGOpad aqua, 10cm x 10cm	20 pcs.
11050	TRIGOpad aqua, 10cm x 10cm	50 pcs.
11030	TRIGOpad aqua comfort, 68mm x 43mm	50 pcs.

Item-N°	Description	DU
Super absorbent wound dressing		
12050	TRIGO Superabsorber, 10cm x 10cm	25 pcs.
12051	TRIGO Superabsorber, 20cm x 10cm	25 pcs.
12052	TRIGO Superabsorber, 30cm x 20cm	25 pcs.
Foam dressing with perforated hydrocolloid layer		
14102	TRIGOcolloid foam, 10cm x 10cm	10 pcs.
Hydrocolloid dressing		
15002	TRIGOcolloid thin, 10cm x 10cm	10 pcs.
15052	TRIGOcolloid thin, 10cm x 10cm	50 pcs.
15005	TRIGOcolloid comfort, 68mm x 43mm	50 pcs.
Transparent film dressing on a roll		
16020	TRIGOfilm, 10m x 5cm	1 roll
16021	TRIGOfilm, 10m x 10cm	1 roll
16022	TRIGOfilm, 10m x 15cm	1 roll